

Ⓢ Installation instructions

## Contents

---

User information	3
Care and cleaning	3
Warning notices	4
Product information	5
Mounting	8
Demounting	13

---

**HEWI support**

+1 646 405 10 32

## Intended purpose

The HEWI hinged seat is designed for use in bathrooms, especially showers. It provides a seat for persons with and without physical disabilities while taking a shower.

## Exclusion of liability

HEWI accepts no liability whatsoever for damage to persons or property that occurs or has occurred as a result of improper mounting or maintenance, particularly if the safety instructions and other points to note listed are not or have not been observed in full. HEWI accepts no liability whatsoever for damage occurring from incorrect cleaning.

## Safety instructions

These instructions are intended for qualified skilled personnel and the user of the installed product. Following installation, the instructions must be made available to the user.

- Follow with the installation instructions and instructions for use.
- In case of questions, problems, uncertainties or if damage occurs, contact the HEWI support service.
- Ensure that the product is installed, maintained and repaired by qualified, skilled personnel only.
- Check the product for damage on acceptance. Report defects immediately.
- Install/use the product only:
  - in accordance with its intended purpose
  - in its original condition without modifications
  - in perfect condition
- Use original spare parts only.
- Only use HEWI fixing materials.
- Ensure that the wall structure and fixings are reinforced as required to permanently withstand the expected load.
- Only use in closed rooms.

## Cleaning



### CAUTION!

### Risk of injury

Improper cleaning can cause damage to the surface and lead to injuries.

- Clean the product as and when required.
- Follow the manufacturer's instructions regarding concentration, contact time, etc.
- Always apply the cleaning product on a cloth; do not apply it directly on the surface.
- Rinse with cold water immediately after cleaning.
- Note the material-specific information on suitable/unsuitable cleaning products.
- Try out all products on an inconspicuous area before use.
- Do not use any products of unknown composition.


### Unsuitable cleaning products

- Abrasive products/objects (e.g. scouring agents, scourer sponge, brushes, steel wool, microfibres, dirt erasers)
- Alkaline, acidic and chlorine-based cleaning products, solvents
- Steam cleaners
- Do not treat metal components with objects containing iron (e.g. scourer sponge, steel wool).

## Disposal

Disposal of the product and packaging is subject to the relevant national regulations.

Tools required

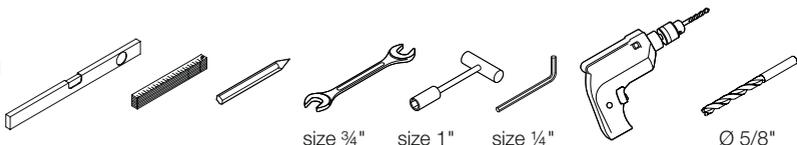

## Warning notices




### CAUTION!

#### Risk of injury

Improperly mounted lift-up seats (loose or at an angle) may lead to injuries due to slipping or falling.

Make sure lift-up seats are installed securely and in a horizontal position.

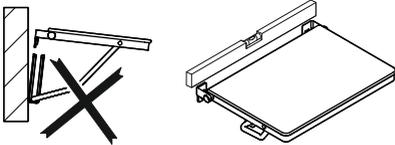


### CAUTION!

#### Risk of injury

Exceeding the maximum load of the lift-up seat may lead to injuries due to falling. Observe maximum load.

Ensure that the wall structure and fixings are reinforced as required to permanently withstand the expected load.


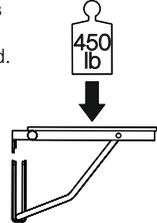


### CAUTION!

#### Risk of injury

Failing or improperly set brakes may cause injuries.

Have the brakes checked and readjusted as required; at least once a month. Do not use any kind of lubricant on the joints or brake discs.

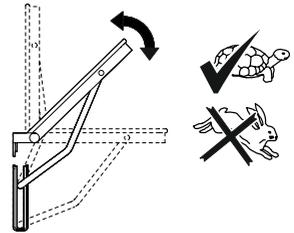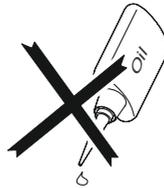


### CAUTION!

#### Risk of injury

Any improper use, e.g. as gymnastic equipment or climbing aid, may lead to injuries due to slipping or falling.

Never stand or jump on the seat.

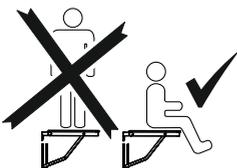


### NOTICE

#### Risk of material damage


The surface of the seat is sensitive to scratches caused by dust or dirt on or near the place of installation. Keep the seat in its packaging for as long as possible.

Once the seat is installed, cover it in its protective foil until all construction work is done and the seat is ready to be used.


900.51.206XA

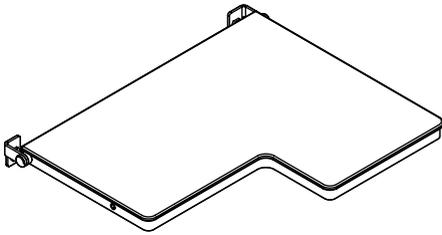


right-sided version

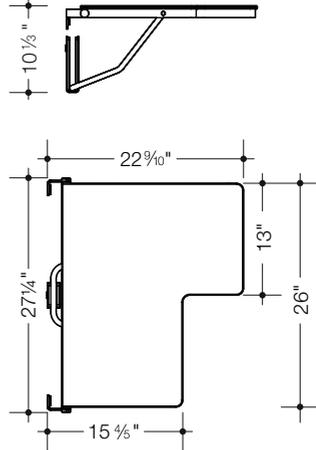

900.51.207XA

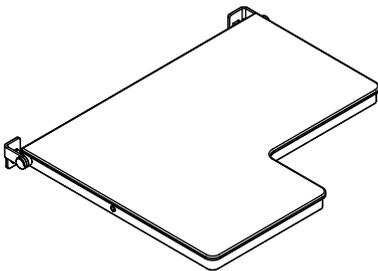


right-sided version

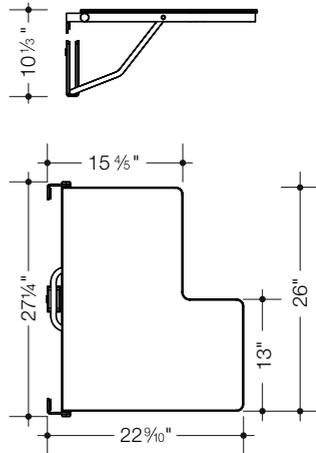


**Product information**


900.51.208XA

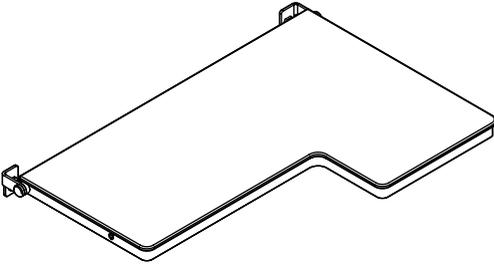


right-sided version

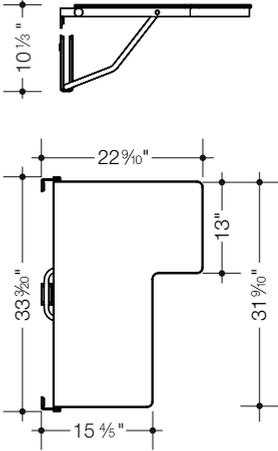




900.51.209XA

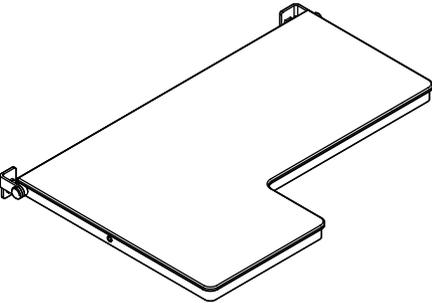




right-sided version

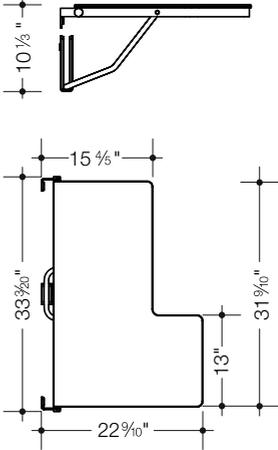

900.51.210XA

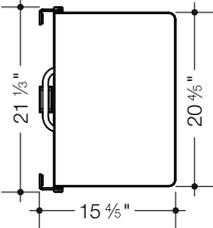
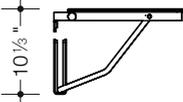
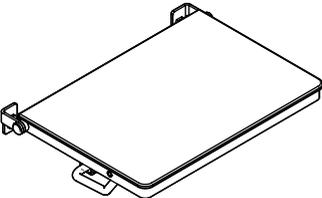


900.51.211XA

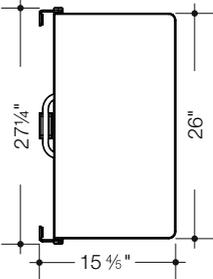
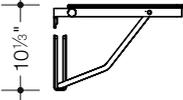
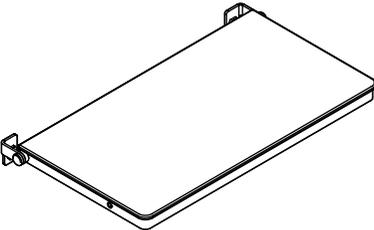


Product information



900.51.212XA

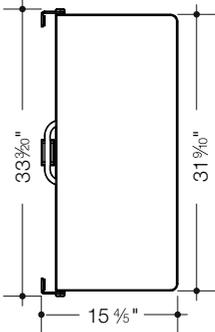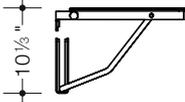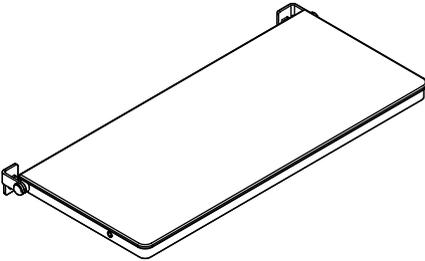


900.51.213XA

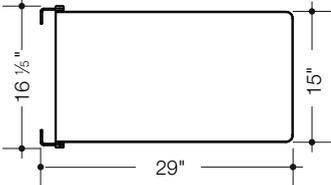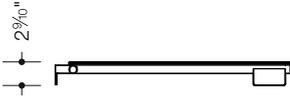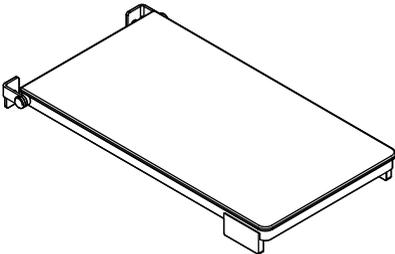

900.51.206...  
 900.51.207...  
 900.51.211...

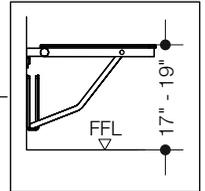
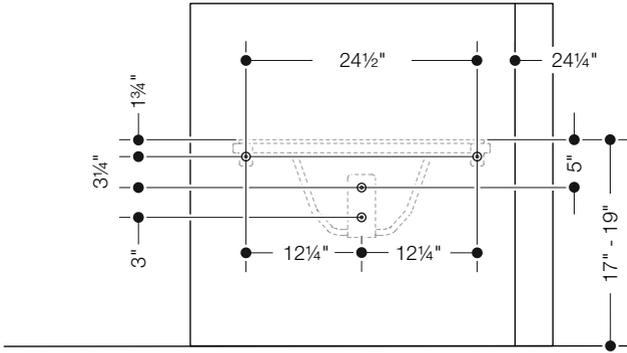


900.51.208...  
 900.51.209...  
 900.51.212...

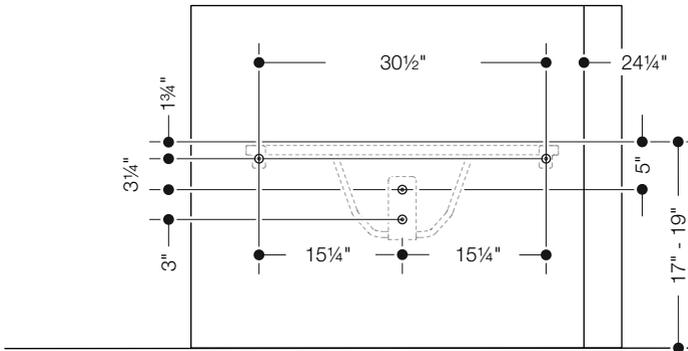





**Mounting**

900.51.210...

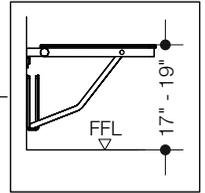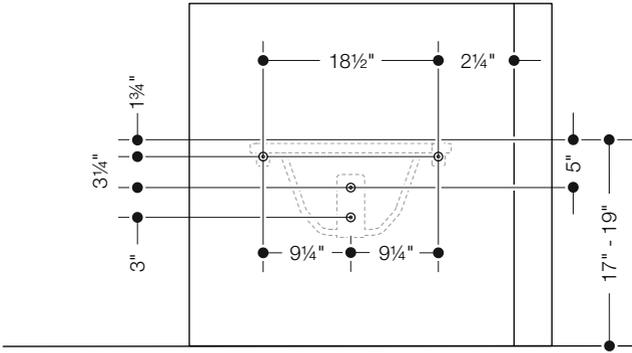


900.51.213...


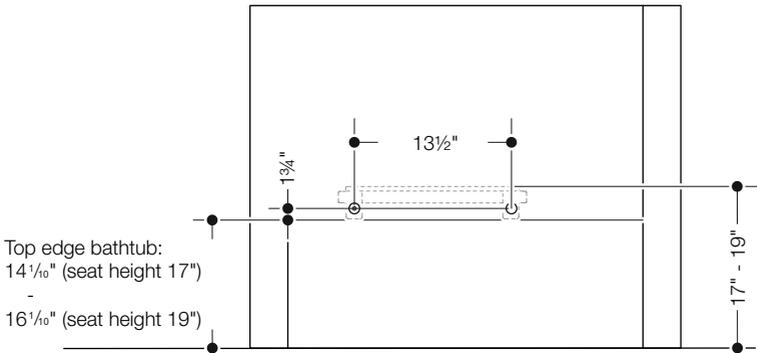


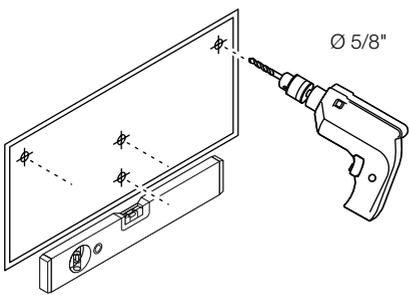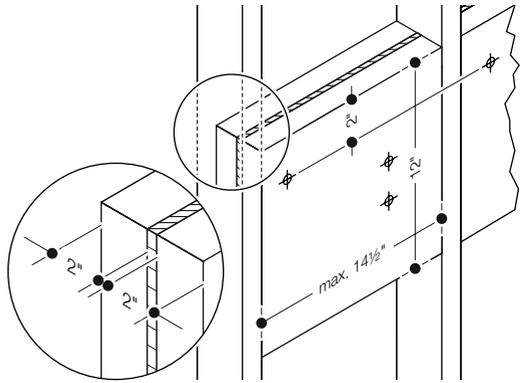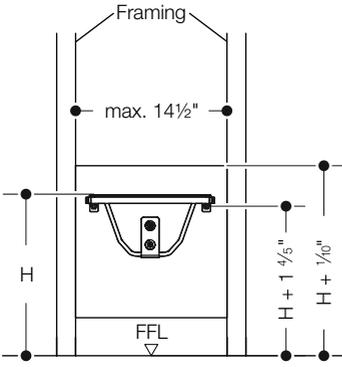

1



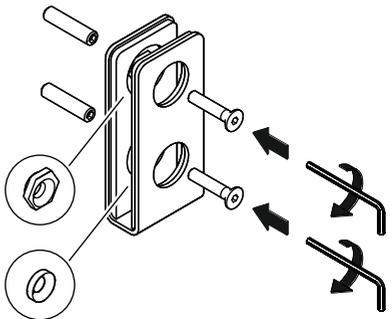





2

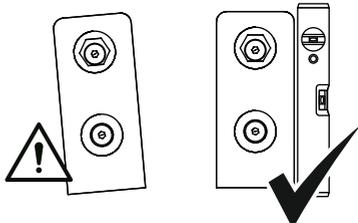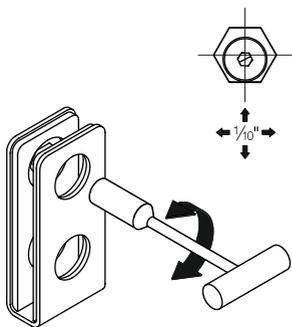

3

Mounting

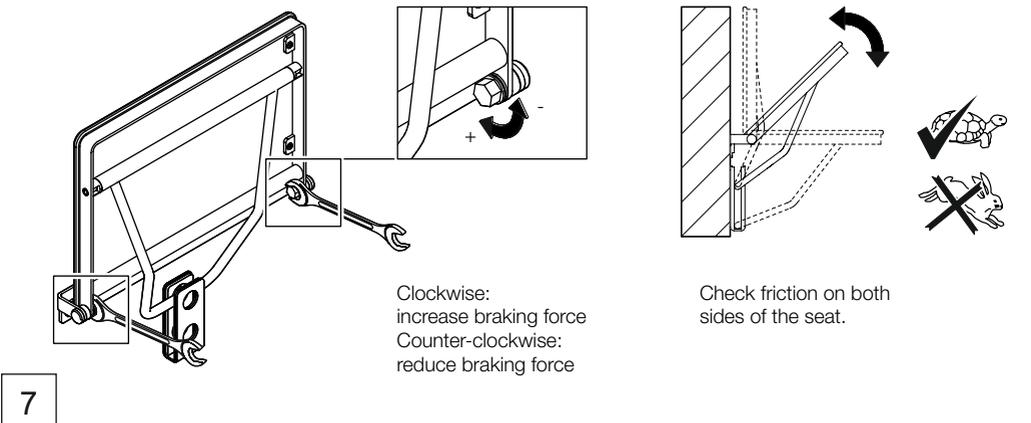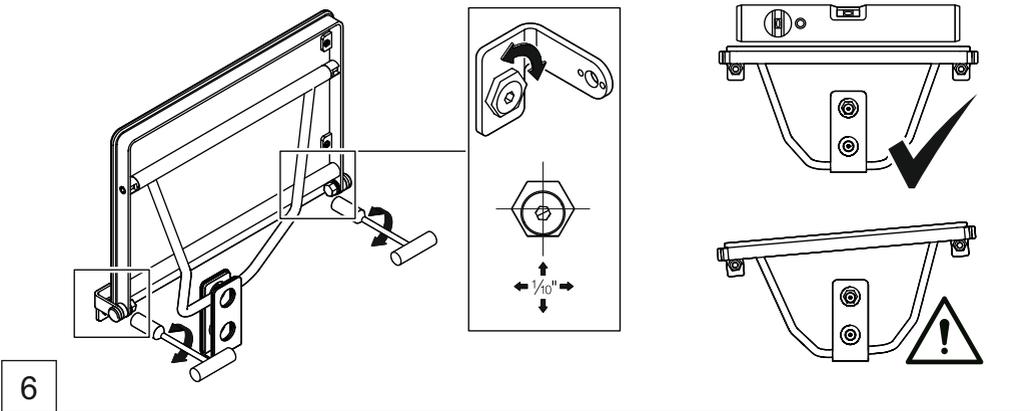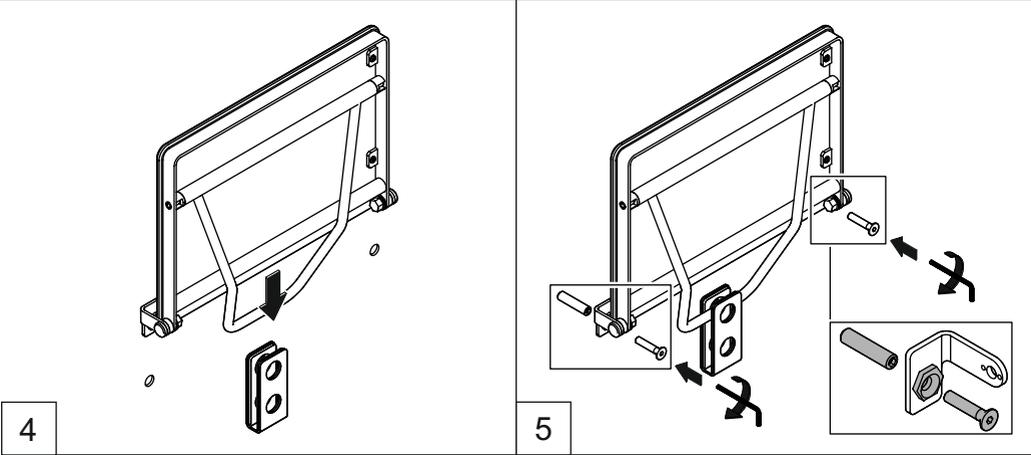

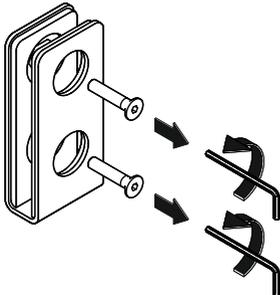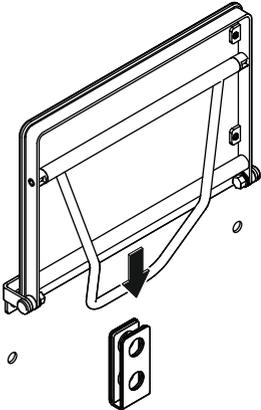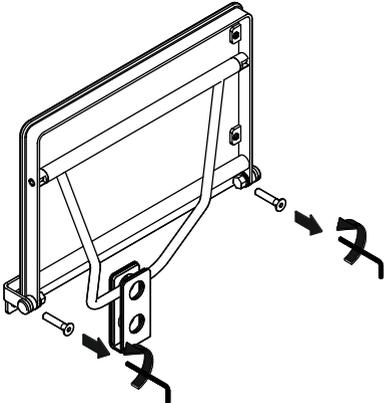



# HEWI

[www.hewi.com](http://www.hewi.com)

## **Deutschland**

HEWI Heinrich Wilke GmbH

Postfach 1260

D-34442 Bad Arolsen

Telefon: +49 5691 82-0

Telefax: +49 5691 82-319

[info@hewi.de](mailto:info@hewi.de)